

Cleaning & Sterilisation Instructions For: Mouth Props

Manual:

1. Pre-disinfect proprietary product follow the manufacturer's instructions.
2. Brush under running water - wearing suitable P.P.E.
3. Dry thoroughly.

Mechanical:

1. Place in the washer/disinfector.
2. Use a pH neutral detergent according to the manufacturer's instructions.
3. Autoclave at 134°-137°C for 3-3.35 minutes.
4. Dry thoroughly with a lint free cloth.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For professional use only. Not available to the general public.



MADE IN TAIWAN



SRU010
(116496UNODENT)



SRU030
(116498UNODENT)



SRU020
(116497UNODENT)



SRU040
(116499UNODENT)



Young Innovations, Inc. 2260 Wendt Street Algonquin, IL 60102
www.younginnovations.com (800) 325-1881



EU REP

Young Microbrush Ireland Ltd. Clogherane, Dungarven
Co. Waterford, X35 VE02, Ireland

UK Responsible Person

Astek Innovations Ltd. 95 Atlantic Street,
Altrincham WA14 5DH United Kingdom Tel: +44 (0)161 942 3900



DD Products & Services Ltd. 6 Perry Way, Witham, Essex, CM8 3SX.
Tel: (01376) 500582. Email: feedback@unodent.com



Trasmore Ltd. T/A DMI, Kellystown House, Kellystown Lane, Leixlip,
Co Kildare, W23 VF9P, Ireland Tel: +353 1 4273700

Symbol	Description	Symbol	Description
	Product reference number		Medical Device
	Sterilizable in a steam sterilizer (autoclave) at temperature specified		Manufacturer
	Country of Origin		Importer
	UK Conformity Assessment		Authorized representative in the European Community
	Non Sterile		Distributor
	Do not use if package is damaged		CE marking Conformity
	Consult instructions for use		Batch or lot code
	Unique Device Identifier		Use By Date